

hypnus Analyzer Installation Instructions

1 System Requirements (Minimum)

- Pentium Dual Core processor
- 2G Memory
- 1G hard disk space
- Operating System : Windows XP/win7/win8/win10

2 Installing hypnus Analyzer

1. Unpack the installation package and the file is shown as:

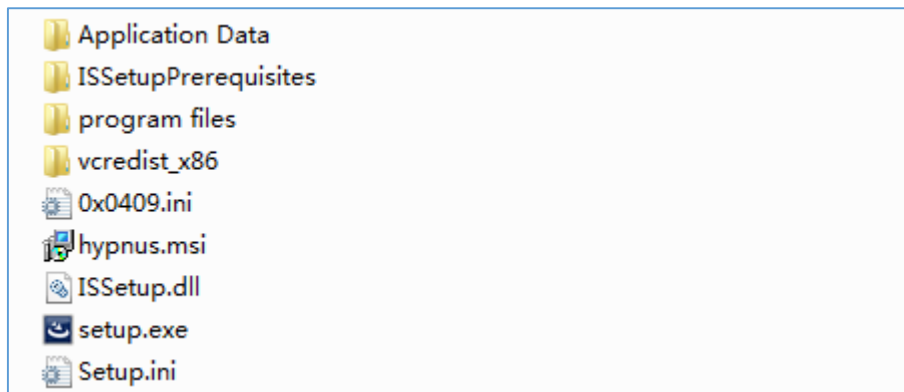


Figure 1

If you had installed hypnus Analyzer 1.2 and earlier version: Uninstall the earlier version and run the “setup.exe” file to install current version.

If you had installed hypnus Analyzer 1.3 and above version: Upgrade the software by running the file “setup.exe” directly.

2. The reminder shown in Figure 2 will pop up if there is not .NET Framework 4.0 in your system. You can install .net framework 4.0 by clicking [Yes] button.

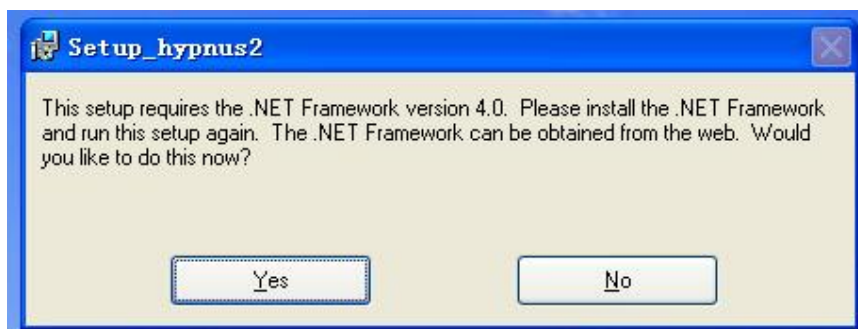


Figure 2

3. A warning shown in Figure 3 will pop up if there is not vs2010 in your system.



Figure 3

You must install the file “vcredist_x86.exe” which is shown as Figure 4 in this situation.

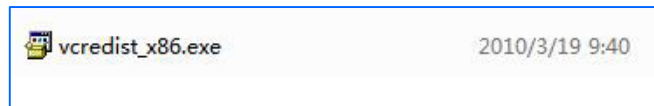


Figure 4

3 Installing PDF Reader

Be sure PDF reader has been installed for the report generated by hypnus analyzer can be read.